



1 Scheduling a Case

Schedule Case



Under Case Management

5 Required Fields:

- ✓ Patient Name
- ✓ DOB
- ✓ Procedure Type
- ✓ Procedure Date
- ✓ Surgeon

Save



2 Pairing a Scope

Pair To Summit Arthroscope 0



Under Case Management



Scan Endcap with iPad Camera

Pair To Summit Arthroscope 1

Button turns white. Indicator turns green. **Scope is paired!**

3 Sharing an Image

05/03/2023
11:26:14
EST



Under Patient Profile



Share



Shares all images with a

05/28/2024
17:19:49
EST

Shared



Now Shared with Patient

